DM-2301

GOLDENS BEAUTY

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DM-2301

GOLDENS BEAUTY

GUIDE PROFESSIONAL MEDICAL COSMETOLOGY SYSTEM PROVIDER

CONTENT

Introduction	11
Technical data	11
Bringing into service	12
Equipment and functions	14
Safety instructions	14
Environment protection	15
Maintenance and care	15

I. TECHNICAL DATA

Model	DM-2301	
Basic configure	Elliptical pillow	
Additional configure	Foot controller, wheel system, hand controller	
Damageable parts	PVC cover, hand controller,	
Technical parameter	Dimension [LxBxH]	200x64x72-95 cm
	Head rest adjustment	-90° ~ 0 ~ 40°
	Max. height	95 cm
	Min. height	72 cm
	Motor QTY	1 (height)
	Net weight	70 kg
	Bed load limit	200 kg

Model	DM-2301	
Basic configure	Pillow	
Additional configure	Foot controller, wheel system, hand controller	
Damageable parts	PVC cover, hand controller,	
Technical parameter	Dimension [LxBxH]	186x68x53-84 cm
	Head rest adjustment	0 ~ 45°
	Anzahl Motor QTY	1 (height)
	Net weight	63 kg
	Bed load limit	200 kg

2. INSTALLATION STEPS

- a) Open the box; place the bed to the proper position.
- b) Connected the power then you can use it, as below picture shows.

3. BRINGING INTO SERVICE

Take the massage bed out of the box and then remove the foil.

Attention: Do not remove the foil with scissors or knives.

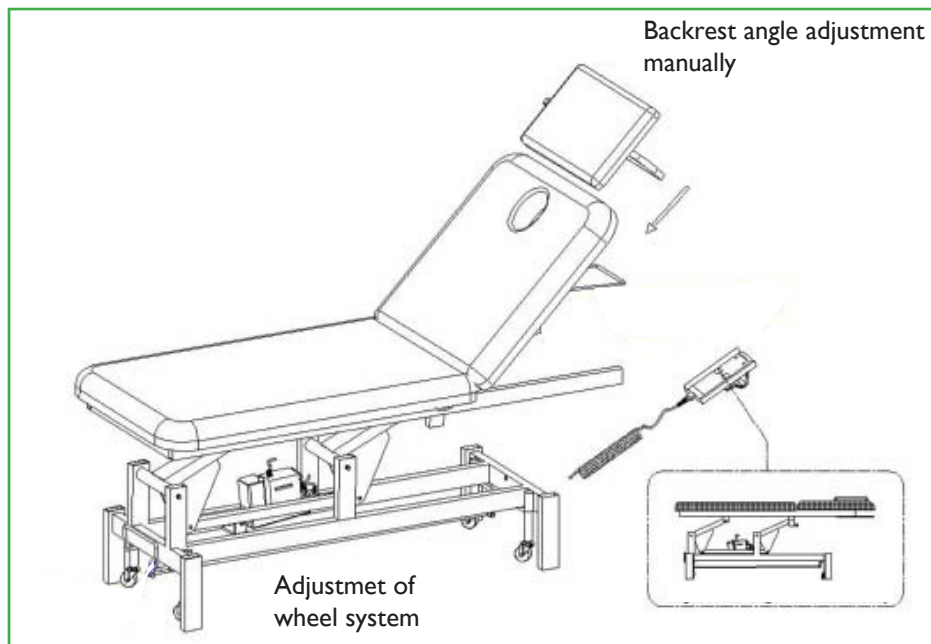
Connect the remote control and plug the power cord into an outlet.

Now you can start adjusting the height by using the remote control.

MODEL DM-2301

MODEL SANA

4. EQUIPMENT AND FUNCTIONS

- Modern massage bed for highest demands
- Simple operation, modern and well-arranged
- Hand controller for adjusting the height
- 1 motor for height adjustment
- Face hole with a pillow for closing
- With wheels for easy moving
- Depending on model, white/ beige leatherette upholstery
- Loop for hand controller on the side
- With security box (When security box is locked, then indication lamp is off, bed cannot be controlled. When security box is unlocked, then indication lamp is on and the bed can be controlled.)
- foot remote control optional available

5. SAFETY INSTRUCTIONS

The equipment is tested in accordance with the current European norms (EN) for this product type and therefore corresponds also with the newest safety regulations. In addition, the equipment meet/complies with all EEC directives that are required in line with the CE identification/labeling.

The manufacturer cannot be held liable for damages or consequential damages, respectively, which result from noncompliance with the following instructions:

- Only certified electricians or service partners may carry out repairs on electrical appliances. Unprofessional repairs can cause/create serious risks for the operator.
- The liability for potential damages is discharged if the product is used inappropriately, incorrectly operated or not professionally repaired.
- This product may only be used with the [provided/original] adaptor – see technical data.
- In cases of malfunctioning during operation, the power plug of the adaptor has to be removed from the power outlet previous to every cleaning and maintenance and after every use.
- Never remove the power plug from the power outlet by pulling on the cord or with wet hands.
- The power cord should never be dragged over wet edges or jammed.
- Children or handicapped persons are not permitted to use the product without supervision.
- Do not operate the equipment in wet surroundings like a bathroom or swimming pools.
- Piercing with sharps objects, like needles, among other things, is prohibited
- Occasionally and depending on use, the product is to be inspected carefully for damages to the synthetic leather cover and the cables.
- In the event of a repair the massage bed is to be shipped only in the original package to prevent additional transport damages.

MAINTENANCE

After every treatment, please use the neutral reagent clear leather, the dust, stains on the plastic cover, in order to prevent premature aging of the leather and the plastic cover.

6. ENVIRONMENT PROTECTION

Raw material recovery instead of waste disposal: The equipment, accessory and packaging should be ecologically recycled.

(Applies only to electrical components)

7. MAINTENANCE AND CARE

Never immerse the equipment in water! The cover was made out of soft and easy-care synthetic leather. Regular dirt and stains can be removed with a solution of mild soap and lukewarm water. Afterwards, rub the cleaned area dry with a soft and clean cloth.

DM-2301

GOLDENS BEAUTY

GUIDE PROFESSIONAL MEDICAL COSMETOLOGY SYSTEM PROVIDER

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